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Use Bonder + Composites that recommend FRC glass fibre posts in their usage, such as:

- CLEARFIL Universal Bond Quick, Kuraray
- PANAVIA F2.0, Kuraray
- CLEARFIL DC CORE PLUS, Kuraray

The root posts for single use must be disinfected and sterilized prior to use.

Root posts are intended for single use only. Unauthorized reuse would present a risk of cross-contamination.

Check all products for intactness before use.

Also observe in the EN ISO 17664-1 Manufacturer's Information the point »Inspection and Functional Check« before initial use.

Years of
Passion for Dental
Excellence

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PREPARATION OF THE ROOT CANAL + INSERTION

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
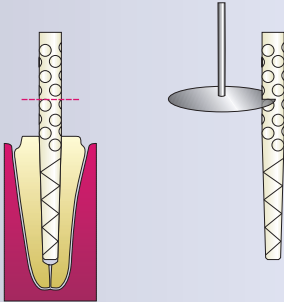
Mark the occlusion height and cut the post extraorally with a fine diamond disc.

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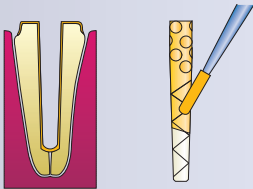
The use of silan for posts is not necessary. **Clean the post with alcohol** and dry it with air. Thinly apply a **dual-cure** bond to the post. Do not polymerise with light. Optional: Then apply the post with composite, do not polymerise with light.

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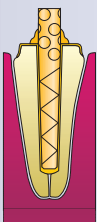
Evenly distribute surplus material on the protruding post and on the cavity. Remove excessive material.

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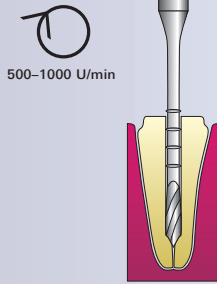Preshaping with enlarging drill REF 42 010 or REF 43 000. For Exatec, use also REF 42 100.

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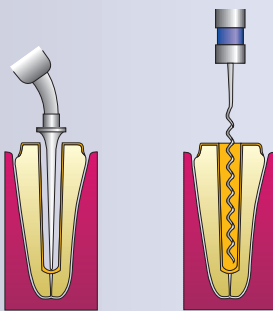
Preparing the root canal: Condition the dentine with the adhesive technique. Observe the Instructions for Use of the according manufacturer.

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Use a lentulo or elongation cannula to fill the root canal with a low-viscous, dual-cure, radio opaque composite.

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
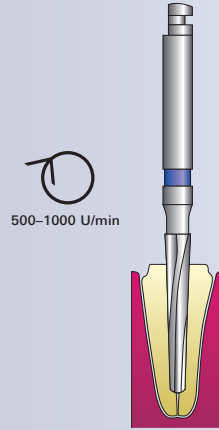
Calibrating the post anchorage with the calibration drill.

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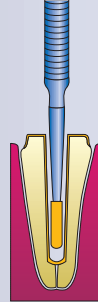
Apply the Primer/Bonder and absorb the surplus with paper tips – do not polymerise Bonder with light.

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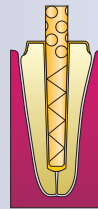Insert the post slowly by turning it slightly into the root canal and keep it in position until the composite is tough enough to hold the post firmly in position.

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Finish the buildup with a viscous core composite and shape it with a diamond grinder when set.