



## Tips and Information

### Our periodontometers

can be cleaned, sterilised (by autoclave or hot air) and disinfected according to all common procedures.

Please see also our Manufacturer's Information acc. to EN ISO 17664.

### PREVENTING LIMESCALE DEPOSITS

#### **1. Machine Processing**

The decalcification of your thermal disinfecter (washer) must be adjusted correctly.



*Image – Limescale deposits and/or water spots*

#### **2. Manual Processing**

Rinse with fully desalinated, deionized water.

### EASY REMOVING of WATERMARKS, CALCIFEROUS RESIDUES and CRUSTIFICATIONS

1. Remove calciferous residues with a soft brush and cleaning liquids.
2. Any calciferous residues can be removed by simply cleaning with acid-containing refresh liquids, e.g. Neodisher N or acetic acid.