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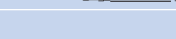
A free »EN ISO 17664-1 Manufacturer Information on Processing« is available within 7 calendar days on request:

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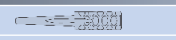
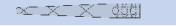
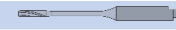
You will find a detailed Instruction for Use on our homepage [www.hahnenkratt.com](http://www.hahnenkratt.com) or you can request it directly from us.

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Use Bonder + Composites that recommend FRC glass fibre posts in their usage, such as:

- CLEARFIL Universal Bond Quick, Kuraray
- PANAVIA F2.0, Kuraray
- CLEARFIL DC CORE PLUS, Kuraray

The root posts for single use must be disinfected and sterilized prior to use.

Root posts are intended for single use only. Unauthorized reuse would present a risk of cross-contamination.

Check all products for intactness before use.

Also observe in the EN ISO 17664-1 Manufacturer's Information the point »Inspection and Functional Check« before initial use.

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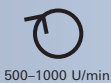
**CLINICAL PROCEDURE**

Please consider also our detailed Instructions for Use as well as the Instruction for Use for the accessory products. Instruments and posts have to be disinfected and sterilized before first use.

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
Preshaping with enlarging drill REF 42 010 or REF 43 000. For Exatec, use also REF 42 100.

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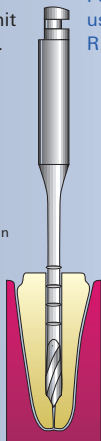
Calibrating the post anchorage with the calibration drill.

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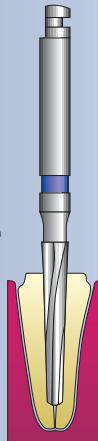
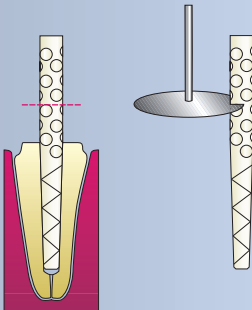
Mark the occlusion height and cut the post extraorally with a fine diamond disc.

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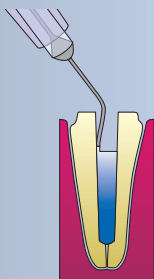
Preparing the root canal: Condition the dentine with the adhesive technique. Observe the Instructions for Use of the according manufacturer.

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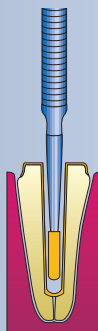
Apply the Primer/Bonder and absorb the surplus with paper tips – do not polymerise Bonder with light.

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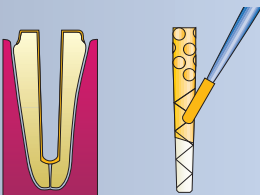
The use of silan for posts is not necessary. **Clean the post with alcohol** and dry it with air. Thinly apply a **dual-cure bonder** to the post. Do not polymerise with light. Optional: Then apply the post with composite, do not polymerise with light.

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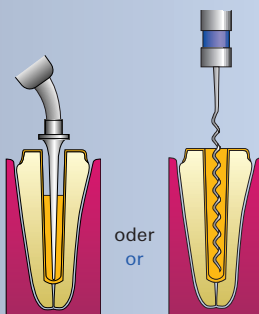
Use a lentulo or elongation cannula to fill the root canal with a low-viscous, dual-cure, radiopaque composite.

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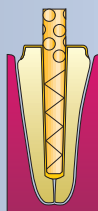
Insert the post slowly by turning it slightly into the root canal and keep it in position until the composite is tough enough to hold the post firmly in position.

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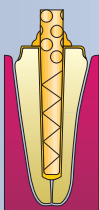
Evenly distribute surplus material on the protruding post and on the cavity. Remove excessive material.

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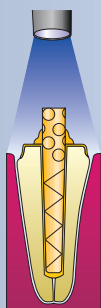Finish the buildup with a viscous core composite and shape it with a diamond grinder when set.